

# ***JMMR Inc.***

***Where Our Innovation is Your Solution.***



## **Simplicity Systems**

### **Blount's KAFO**

Introducing a new Orthotic treatment for Blounts disease and other pediatric lower limb deformities. The JMMR Blount's KAFO allows normal daytime activity while applying corrective forces to the patients limb.

The JMMR Blount's KAFO is a custom molded KAFO with a unique dynamic correction force system to treat leg deforming problems. The JMMR KAFO has a patented distal posterior swivel hinge that allows the molded tibial section to be pulled with an elastic force strap to provide the necessary dynamic corrective forces to correct the limb deformity. The corrective forces are applied distal to the knee center. This allows normal ambulation and full knee and ankle joint mobility. Correction can be anticipated in significantly less time than traditional approaches

*Patient Compliance will no longer be a problem. Patient and parent friendly.*

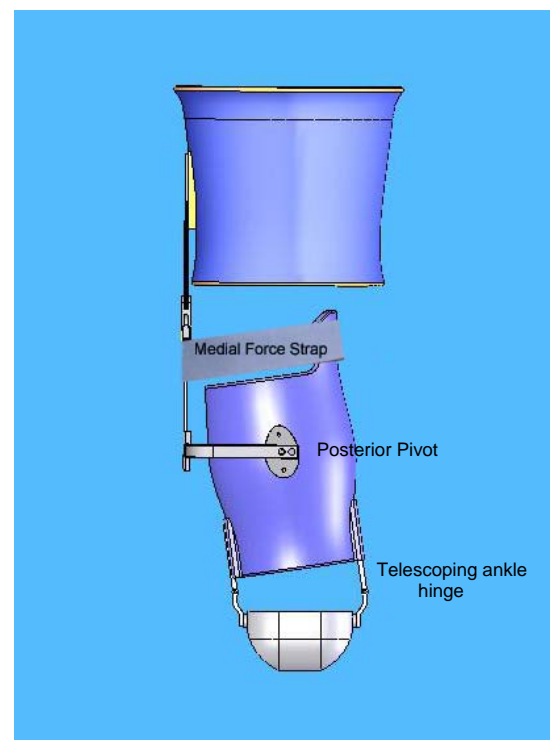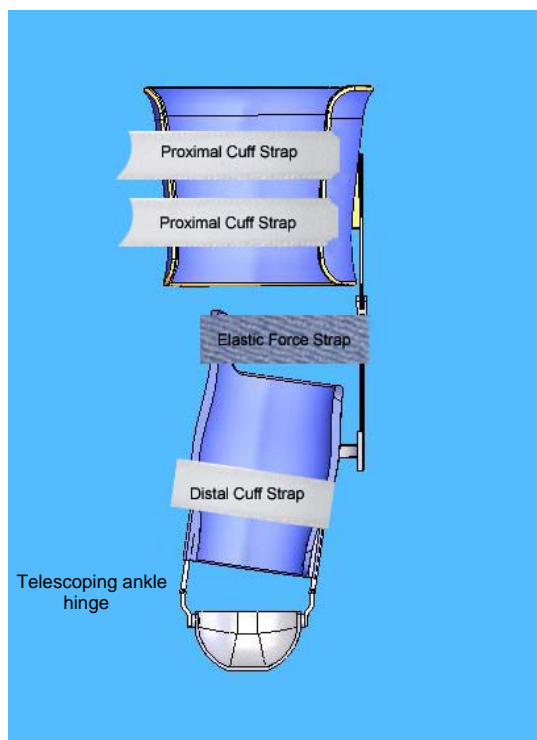

# Actual Cases: Typical Results



30 Month old Female Diagnosed with Blount's Disease presented with 15 degree angular deformity corrected to 6 degrees in 20 weeks

24 Month old male Diagnosed with Blount's Disease presented with a 14 bilateral angular deformity corrected to 5 degrees in 22 weeks



The JMMR Blounts KAFO is patented and available only through JMMR Inc. The orthosis is central fabricated from your cast and measurements. The turn around time is usually 10-14 days. We recommend you cast your patient with M-Pact OCL Flex Cast Tape (0292f (2'') or 0293f (3'')). This is a flexible fiberglass wrap that is removed with a sharp serrated scissor. This material cures fast and has an excellent memory. Mark all boney areas, indicate knee center, fibula head, medial and lateral malleolus and navicular. Do not leave the stockinet in the cast



Inset Photo  
The telescoping lateral member of the ankle joint insures proper contact of the foot to the walking surface during dynamic correction of the tibial section

***We guarantee results or we will take back the brace for a full refund.***

JMMR Inc. 2 Wilsey Square, Ridgewood, New Jersey 07450  
 800-236-5166 Fax 201-612-1323 Email: [simplicitysystems@earthlink.net](mailto:simplicitysystems@earthlink.net)  
 For additional information visit our website: [www.blountsdisease.net](http://www.blountsdisease.net)